

## **iWhite instant interdental whitener:**

### **1- Brand name**

iWhite Instant Interdental Whitener

### **2- Description & Properties**

The iWhite Interdental Whitener is an innovative interdental whitening device loaded with teeth whitening actives to eliminate staining and to whiten the interdental spaces effectively. The formula is based on the whitening 3- technology:

#### *Action 1: Physical whitening*

The delivery device is therapeutically coated with small cleaning particles which will physically eliminate the interdental staining.

#### *Action 2: Active whitening*

Active whitening based on PAP. PAP is a new whitening action substance for effective and deep whitening of the teeth and is absolutely safe for tooth enamel and gums. PAP also inhibits the growth of bacteria in the mouth, giving plaque no chance.

#### *Action 3: Optical whitening*

The optical whitener, blue Covarine, alters the way in which light reflects on and between the teeth, giving a visibly whiter appearance immediately.

### **3- Indication**

The iWhite Interdental Whitener is used to eliminate staining and to whiten the interdental spaces effectively. The iWhite Interdental Whitener touches places that the toothbrush or mouthwash can't reach and can be used on a daily base.

### **4- Warnings and Precautions**

- Not recommended for children under 14 years.
- Do not use this product if you suffer from gum problems.
- Discontinue the treatment if you experience pain or excessive sensitivity.
- Use of the product is inadvisable if you have allergic reactions to one of the ingredients.

### **5- Instruction for use**

1. Use about 20 cm of floss, leaving 3-5 cm to work with.
2. Gently follow the curves of your teeth by using zig-zag motion.
3. Slide floss up and down against the tooth surface and under the gum line. Floss each tooth thoroughly with a clean section of floss.

### **6- Storage condition**

- Store in a cool dry place between 2°C and 25°C.
- Keep away from heat and direct sunlight.
- Keep out of reach of children.

### **7- Packaging**

Each box contains floss for 125 treatments

### **8- License Holder**

Sylphar NV/ Belgium

**9- Marketing Authorization Holder in IRAN**  
Koushan Pharmed